

# HALO TEST STEP BY STEP



**Open your test packet and take out the contents.**



**Check the test tube fill line is 2ml and then unscrew the cap.**



**Take the funnel out of the packet and screw it onto the test tube.**



**Spit into the funnel until the saliva is above the 4ml line.**



**Once full, unscrew the funnel, and screw the lid tightly back on.**



**Open the alcohol wipe, wipe the test tube and our hands.**



**Put the test tube into the plastic biohazard packet.**



# HALO TEST STEP BY STEP



**Peel off the yellow sticker and seal the packet shut.**



**Put the plastic biohazard packet into the cardboard packaging.**



**Peel back the yellow sticker and seal the cardboard packaging shut.**



**Place your cardboard package into the COVID test bin.**

